

Oticon wireless accessories

Everything you need to **stay connected**



oticon
life-changing technology

Make everyday tasks easier

Oticon wireless accessories are a comprehensive and easy-to-use range of devices developed to improve your listening and communication experiences with your hearing aids.

Our wireless accessories help you get the most out of your Oticon hearing aids.* For example, you can stream the sound from your TV, smartphone, music player, PC or other devices to your hearing aids - just like wearing wireless headphones.

Control your hearing aids with Oticon ON App

Oticon ON App lets you discreetly adjust the volume, switch between programmes, use HearingFitness™, access the Internet of Things, and much more.

* Any Oticon hearing aid with 2.4 GHz Bluetooth® low energy technology.



Connecting your child to the world

Oticon's hearing aids for children are much more than hearing aids - they empower your child to connect seamlessly to the modern world.

In addition to the comprehensive range of wireless accessories offered to adults, we now introduce the EduMic - designed to solve many of the daily hustles and bustles of a school day.

Read more on page 14



Make clear, safe and hassle-free calls

With 2.4GHz Bluetooth low energy technology your Oticon hearing aids can connect directly to iPhone® or any other modern smartphone via ConnectClip.



Hands-free calls for any smartphone

With ConnectClip, it's easy to make hands-free calls with an iPhone, Android™ or any other smartphone.

ConnectClip transmits the caller's voice directly to your hearing aids while the built-in microphone clearly transmits what you say to the caller. ConnectClip transforms your hearing aids into a high-quality wireless headset enabling you to:

- Make and receive hands-free calls in the car or on the go - streaming to both ears
- Wirelessly stream music, videos or other sound from your smartphone to your hearing aids
- Control the volume and change programmes remotely



Your hearing aid is a Made for iPhone hearing aid

- Connect directly to your iPhone, iPad®, or iPod touch®
- Turn your hearing aids into stereo headphones
- Stream calls, music, videos or other sound wirelessly to your hearing aids
- Use your iPhone to control volume or change programmes



Apple, the Apple logo, iPhone, iPad, and iPod touch are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc. Android, Google Play, and the Google Play logo are trademarks of Google LLC.




Hear the person talking with more clarity

Even with the best hearing aids, it can sometimes be difficult to clearly hear what someone is saying – such as in noisy environments, when attending a lecture, talking on your home phone or over the computer.

The wireless ConnectClip can help you in all these situations by streaming sound directly to your hearing aids. This will make it easier to listen to the person you want to hear while further reducing the background noise.

Hear speech at a distance or in noise

- Use ConnectClip as a remote microphone by handing it to the person you want to hear better
- ConnectClip streams the speaker's voice directly to your hearing aids
- Use Oticon ON App to control the speech volume and the level of background noise in ConnectClip
- Range: up to 20 metres

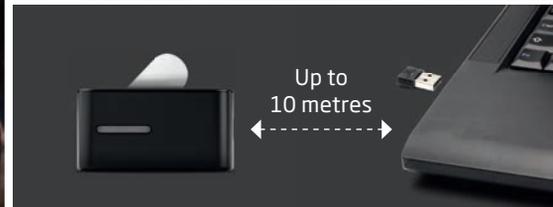


Chat over the computer

- Pair ConnectClip directly to your computer or use Oticon's USB dongle
- Use ConnectClip to stream the sound directly to your hearing aids when chatting over Skype, Lync, etc. or when watching videos
- Range: up to 10 metres

Talking on your landline phone

- Connect Phone Adapter 2.0 to your existing phone system
- Use ConnectClip to answer the call
- Hear the caller's voice directly in your hearing aids
- Range: up to 30 metres

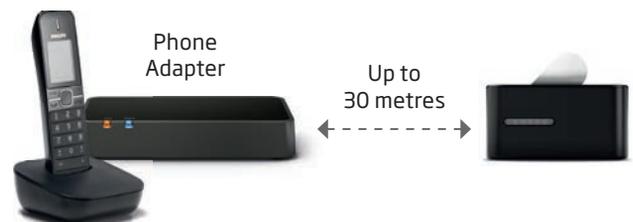



Control your hearing aids remotely

If you want to control the volume, change programmes or simply turn off your hearing aids with complete discretion, you can use any of the three remote control options: Oticon ON App, ConnectClip or Remote Control.

Get sound delivered **straight** to your hearing aids

TV Adapter 3.0 lets you enjoy your favourite TV programme or listen to music from your stereo directly in your hearing aids - just as if you were wearing wireless stereo headphones.

With TV Adapter you can:

- Clearly hear the TV or listen to music by streaming the sound directly to your hearing aids
- Choose the level you want to hear while leaving the TV at a volume that is comfortable for others
- Pair up to four TV Adapters to your hearing aids and switch between different audio sources
- Use Oticon ON App to control the volume of the streamed sound remotely
- Range: up to 15 metres



Boost your hearing aids with Oticon ON App



Control your Oticon hearing aids easily and discreetly with Oticon ON App

The Oticon ON App is available for iPhone and Android smartphones, and opens up a wide range of features and functionalities:

- Switch between programmes
- Adjust volume
- Check battery status
- Locate your hearing aids
- Access to the Tinnitus SoundSupport™ feature
- Use HearingFitness
- Connect to the Internet of Things

Read more on oticon.ca/app

Boost your hearing when you need it most

In an everyday situation where you need more help to make speech stand out from noise, you can use the new booster function in the Oticon ON App to give your hearing aids an extra boost.

Getting started

1. Download the free Oticon ON App from the App Store or Google Play
2. Pair your Oticon hearing aids with your iPhone, iPad, Android™ smartphone or tablet
3. You're all set!



Apple, the Apple logo, iPhone, iPad, and iPod touch are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc. Android, Google Play, and the Google Play logo are trademarks of Google LLC.

Enjoy **remote communication** with your hearing care professional

Although nothing can replace a face-to-face consultation for more major issues, modern technology makes it easy to carry out routine adjustments to your hearing aids without the need for you to go into the clinic.

Oticon RemoteCare App enables you to:

- Get expert hearing aid follow-up consultations in the comfort of your own home - or wherever else is convenient*
- Save time and money on travelling to the hearing care clinic for minor hearing aid adjustments
- See, hear, talk and text with your hearing care professional during your remote appointment
- Get your hearing aids adjusted remotely to assist you in getting the best out of them
- Test your new hearing aid settings while you are talking with your hearing care professional



Download Oticon RemoteCare App from the App Store and Google Play



The availability of this service may vary. Please contact your local hearing care professional for more information.

* Technical requirements: You need a stable Internet connection, iOS device running iOS 12 or later or a selected Android device running OS 8.0 or later. For information on compatibility please visit oticon.ca/compatibility

HEARING
FITNESS™



Fulfill your hearing potential, maintain your health

Oticon helps you get the best out of your hearing aids. Just like an exercise app for the ears, HearingFitness in Oticon ON App gives you encouragement on ways to hear better, protect your hearing, and improve your health.

HearingFitness receives data from your hearing aids and analyses current sound environments, total daily hearing aid use and historical use data. In this way, HearingFitness gives you a clear overview of daily, weekly and monthly performance and lets you compare it to the goal you set for your hearing aid use.*

* Oticon HearingFitness is constantly evolving. Download the very latest version and functions from the App Store or Google Play.



Maximizing the opportunities in life

Hearing key information can be a challenge. Sometimes it's when trying to hear someone speak over background noise or in a crowd. Other times, it's hard to hear at a distance, such as in church, at the theatre, during education, and when playing sport.

EduMic solves these issues by transmitting an individual voice directly into the user's hearing aids. It is a discreet, remote microphone that's easy to use. What's more, it features a telecoil to pick up audio from public audio loop systems and transmit to the hearing aids.

EduMic also transmits audio from all sorts of audio devices like computers, smart boards, tablets or smartphones - connecting via a standard 3.5 mm headphone jack.

Helping your child hear in noisy situations

Children with hearing loss can be especially challenged in school, where noisy listening environments can make it hard to follow the class. This is where EduMic offers great learning support by giving direct access to the teacher's voice.

Outside school hours, EduMic can be your invaluable link to your child. At home, it doesn't matter if you have appliances running, the TV on, or siblings bickering - EduMic will bring your voice directly to your child's ears. What's more, you can use EduMic at the playground, out shopping, and in any other noisy environments.



Oticon wireless accessories are compatible with:

Oticon Opn S
Oticon Opn Play
Oticon Xceed
Oticon Xceed Play
Oticon Opn
Oticon Ruby
Oticon Siya

For more information on products and compatibility between devices, talk to your hearing care professional today or go to oticon.ca/connectivity

Contact your local hearing care professional:

Manufactured by :

Oticon A/S
Kongebakken 9
DK-2765 Smørum
Denmark
www.oticon.global

Imported and Distributed by :

Audmet Canada Ltd
1600-4950 Yonge St
Toronto, ON M2N 6K1
www.oticon.ca

oticon
life-changing **technology**